

# PROPHY AID™

A SIMPLE & EASY PATENTED PRODUCT TO HELP POLISH YOUR PATIENT'S TEETH.

*Easy step-by-step instructions...*




- < 1. PLACE DISPOSABLE PROPHY PASTE CUP UPSIDE DOWN ON A HARD SURFACE.

- > 2. HOLD THE PROPHY AID AND FIRMLY PRESS DOWN OVER DISPOSABLE PROPHY PASTE SO THAT IT FITS TIGHTLY INTO THE PROPHY AID.



- > 3. PLACE THE PROPHY AID ON THE SMALL SUCTION APPROX. 2-3 INCHES AWAY FROM THE TIP. BEND SUCTION TIP FOR RETRACTION.

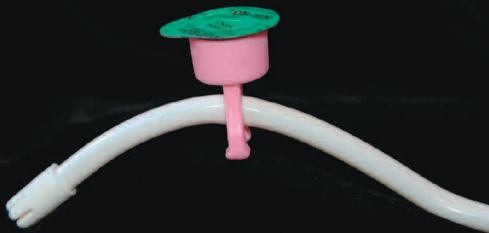

- > 4. HOLD THE PROPHY AID BY PLACING YOUR THUMB AND POINTER FINGER ON THE FINGER RESTS OF THE PROPHY AID



*Now polish & suction at the same time.  
Simple & Convenient!*

CALL TODAY FOR YOUR  
INTRODUCTORY OFFER AT  
**1-888-HYG-AID8**

OR VISIT US AT  
**WWW.PROPHYAID.COM**